

WOUND CARE

Learn how to easily apply WIMBA Wound Care in three simple steps.



STEP 1

Primary bandage layer

Apply the first protective bandage layer directly over the limb to create a clean, even base for further dressing application.



STEP 2

Wound Care placement and fixation

Position the WIMBA Wound Care over the selected area and secure it in place, ensuring appropriate fit, stability, and pressure offloading.



STEP 3

Final bandage application

Cover the WIMBA Wound Care with the outer bandage layer to stabilise the dressing and complete the protective bandage construct.